

ALHYDRAN : Product range



ALHYDRAN 30ml / 1 fl oz



ALHYDRAN 100ml / 3½ fl oz



ALHYDRAN 250ml / 8½ fl oz

ALHYDRAN : Frequently asked questions

In what quantities is ALHYDRAN available?

ALHYDRAN is available in packs of 30ml, 100ml, and 250ml.

Are there any side effects from ALHYDRAN?

No side effects have been discovered as yet.

Is it possible to use it too much or for too long?

ALHYDRAN can be used permanently (over several years).

How should ALHYDRAN be stored?

- Store at room temperature and no higher than 25°C.
- You can use ALHYDRAN until the expiry date given on the pack.

Where can I get ALHYDRAN?

ALHYDRAN is only available through specialists and general practitioners.

When should I not use ALHYDRAN?

- You must not use ALHYDRAN if you have a pronounced allergy to one of the components.
- ALHYDRAN must not be applied to an open wound or open skin.
- Do not use it on infected skin.
- Do not use it on the eyes.

How can I combine ALHYDRAN with other therapy(s)?

ALHYDRAN can be used in combination with other (pressure) therapy. Apply ALHYDRAN thinly on the skin. If necessary remove excess gel crème after 15 minutes with a dry cloth. Wait for 15 minutes and put on pressure garments or silicone bandages. After removing the pressure garments or silicone bandage, treat the skin with ALHYDRAN until it is totally absorbed by the skin.

For more information, see the instructions.



BAP Medical BV, the Netherlands
P.O. Box 531
NL-7300 AM Apeldoorn
phone +31 (0)55 355 25 80

www.bap-medical.com
info@bap-medical.com

BAP Medical BVBA, Belgium
Bornstraat 6
B-8800 Roeselare
phone +32 (0)51 24 00 07

ALHYDRAN

medical after care for the skin

Medical gel-cream for the treatment
of damaged skin immediately
after wound closure

After care for burns,
skin transplants,
and chronic wounds

•
Reduces itching

•
Prevents scars

•
Hydrates dry, sensitive,
and vulnerable skin

•
After care for damaged skin after
radiotherapy

The skin is the largest organ of the body and its most important task is to protect the body against external influences. If the skin is damaged, a scar begins to form and this damaged part of the skin no longer functions properly. This happens, for example, with burns, chronic wounds, dry skin, or other skin damage such as radiotherapy, laser treatments and peelings. This vulnerable skin lacks the qualities to protect the body properly. Therefore, this damaged skin requires extra care and treatment.

What is **ALHYDRAN** ?

ALHYDRAN is a concentrated, medical, powerful moisture regulating gel-cream which combines the strength of:

- freshly processed, pure Aloë Vera Gel (Aruba Aloe Barbadensis) and
- high quality oils and ingredients such as Jojoba oil, Vitamin E, and Vitamin C.

ALHYDRAN does not contain added scent, is gentle on the skin, and low allergenic. The delicate perfume of ALHYDRAN is the one of freshly harvested aloe vera gel, as main ingredient of the medical gel creme. ALHYDRAN has not been tested on animals.

The therapeutic and hydrating properties of Aloë Vera Gel for vulnerable skin have been well known for years. The Aloë Barbadensis from Aruba has been specially selected for ALHYDRAN because Aruba has the ideal climate for the Aloë Vera so that ALHYDRAN offers the highest concentration of effective gel.

Clinically tested

ALHYDRAN has been developed and clinically tested at length for the after-care of burn patients in both the Clinic of Plastic and Reconstructive Surgery and the Burns Centre of the University hospital of Ghent (Belgium).

What can you use **ALHYDRAN** for?

After-care for burns:

ALHYDRAN has a therapeutic effect on the newly closed skin during the after-care of burns (both deeper burns and first degree burns) and skin transplants. It reduces itching and redness and ensures that the skin becomes more flexible. It helps to prevent scars.

Prevention of scar formation

By means of early application of ALHYDRAN, immediately after wound closure, the skin remains flexible and gets an optimal treatment to prevent scar formation. When dealing with abnormal scar formation (hypertrophic, keloid) ALHYDRAN is an excellent completion of existing scar treatments. ALHYDRAN also reduces itching and softens the colour of scars.

Treatment of the skin immediately after wound closure:

ALHYDRAN is designed for the treatment of newly closed skin from acute and chronic wounds. ALHYDRAN softens and hydrates the skin and helps to prevent the formation of (irritating) scars.

Treatment of the (very) dry skin.

ALHYDRAN hydrates and softens the skin. Moreover, it reduces itching and treats the (very) dry, irritated, and vulnerable skin.

Other indications ALHYDRAN:

- after care of damaged skin after radiotherapy
- reduction of itching
- after care vulnerable skin after laser treatments
- after care vulnerable skin after peelings

The properties of **ALHYDRAN** ?

ALHYDRAN helps to treat and prevent complaints such as pain, itching, and scars.

ALHYDRAN helps to produce a general improvement in the cosmetic condition of the skin, to improve fluid regulation, and to reduce redness.

ALHYDRAN helps to make the skin softer so that it functions better.

How do I use **ALHYDRAN** ?

- Apply ALHYDRAN at least three times a day or whenever the skin itches, is dry, or feels irritated.
- Apply ALHYDRAN thinly and rub it in slowly until it is absorbed completely by the skin.
- Do not apply ALHYDRAN to an open wound.
- You must not use ALHYDRAN if you have a pronounced allergy to one of the components.

ALHYDRAN can be used perfectly well together with pressure suits, silicone sheets, and/or silicone pressure suits.

Before putting on:

Apply ALHYDRAN thinly on the skin. If necessary remove excess gel crème after 15 minutes with a dry cloth. Wait for 15 minutes and put on pressure garments or silicone dressings.

After wearing pressure:

Treat the skin with ALHYDRAN after removing the pressure stockings/garments or silicone bandages.

